



Mountain View Medical Supply  
 5076 W. 58th Ave.  
 Arvada, CO 80002  
 303-455-1300  
 800-873-7121  
 MVMSinc.com

**Activon**  
 100% Manuka Honey  
 The Natural Wound-Healing Solution



medi. I feel better.

# Activon

## Manuka Honey

Activon Manuka Honey, unique to New Zealand, is made from the nectar gathered by bees from the *Leptospermum Scoparium* plant.



**100 %**  
MEDICAL GRADE  
**MANUKA**  
**HONEY**

Activon Manuka Honey promotes four essential wound healing functions, which are often found individually in other types of dressings.

## Antimicrobial

- Manuka Honey has a pH of ~3.9 (acidic), creating an inhospitable environment for bacteria growth.
- Low pH levels improve tissue oxygenation which increases resistance to infection and promotes healing.
- Super saturated sugar content limits water availability for bacteria.
- Manuka Honey contains high levels of Methylglyoxal, yielding powerful antibacterial properties.


## Odor Control

- Eliminates the odor at the source (bacteria), rather than masking it.
- Removing malodor improves patient confidence and mental well-being.



## Autolytic Debridement

- 100% Manuka Honey creates a moist wound environment, facilitating autolysis by drawing fluid/exudate from the wound through powerful osmotic action, while simultaneously rehydrating devitalized tissue.
- Safe and effective debridement and desloughing

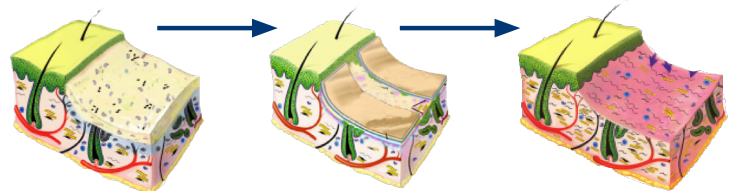

## Granulation and Epithelialization

- Promotes growth of granulation tissue by drawing up nutrients from the interstitium
- Provides glucose to cells that promote epithelialization across the wound

# Activon Tube

## Description

Activon® Tube contains 100% Manuka honey.

## Indications

Activon® Tube may be applied to any type of wound and is especially useful for cavity wounds. Can be applied directly to a wound bed or on a primary dressing.



Product	SKU	Sizes	Pack Sizes	HCPCS* Code
<b>Activon® Tube</b> 100% Medical Grade Manuka honey	CR3830	25g	12	A6250


# Activon Tulle

## Description

Activon® Tulle is a knitted viscose mesh impregnated with 100% Manuka honey.

## Indications

Activon® Tulle is ideal for shallow wounds that need debriding of necrotic tissue.

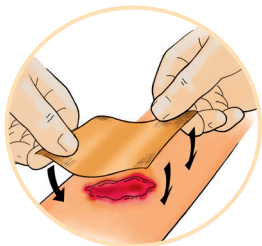

**100%**  
MEDICAL GRADE  
MANUKA  
HONEY

Product	SKU	Sizes	Pack Sizes	HCPCS* Code
<b>Activon® Tulle</b> Knitted viscose mesh with 100% Manuka honey	CR4135 CR4136	2in x 2in 4in x 4in	5 5	A6402 A6402

\*Code usage should be verified by responsible parties in accordance with all published guidelines.

# PREPARE

Compress with confidence and prepare wounds with autolytic debridement

# 1

# HEAL

Create optimal wound healing environment and accelerate closure with accurate compression

# 2

# PREVENT

Sustain healthy skin and integrate lifestyle compression to prevent reoccurrence

# 3


## Positively evaluated in hundreds of patients

### Bilateral Venous Leg Ulcer

Clinical signs of infection: pain, offensive odor and excessive exudate.

Treated with Activon® Tulle and Activon® Tube.

Reduction in pain, exudate and odor by 2nd dressing change. Almost healed within 6 months.

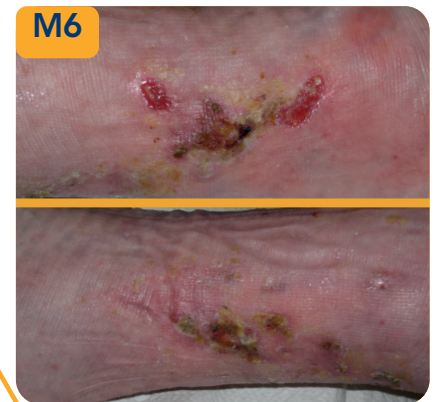

medi USA  
6481 Franz Warner Parkway  
Whitsett, NC 27377  
T 800-633-6334  
F 888-570-4554  
mediusa.com

**Adv**ancis  
Medical  
[www.advancismedical.com](http://www.advancismedical.com)