



**PREMIUM
OVERNIGHT
PROTECTION**

**100%
SATISFACTION
GUARANTEED**

Attends®

.....PREMIER.....

Attends® Premier Underwear Sizing Charts (ALI-UW)

Finding the right size is extremely important, whether you're using the product yourself, or caring for a loved one. **Proper sizing can help to prevent leaks, skin irritation and improve overall comfort.** Use the chart below to see the recommended size of product to use.


Sizing is not a perfect science; sizes provided below are only recommendations.

If you have any questions about sizing or about Attends® Premier products, please call our **Customer Care Team at 1.800.4.Attend (1.800.428.8363).**



UNDERWEAR SIZE		WEIGHT	WAIST SIZE	WOMEN'S DRESS SIZE*
MEDIUM	ALI-UW20	120 - 175 lbs	36" - 44"	8 - 14
LARGE	ALI-UW30	170 - 210 lbs	44" - 58"	16 - 20
X-LARGE	ALI-UW40	210 - 250 lbs	56" - 68"	22 - 24

*Sized to coordinate with women's dress sizes to make it fast and easy to find your perfect fit.


How To Apply Attends® Premier Underwear

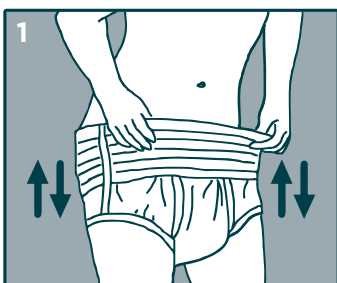

1 Remove the underwear from the bag and unfold. Apply to the body just like regular underwear.

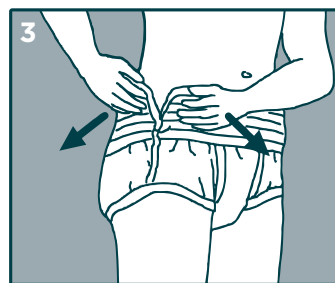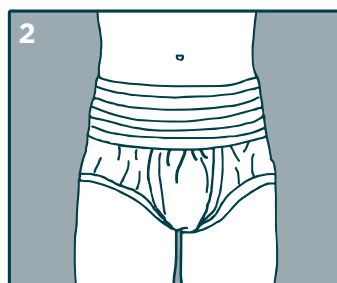

3 When removing, pull the tear-away seam on either side.

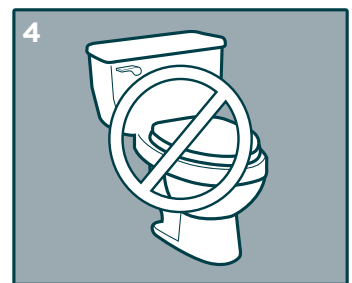

4 This product is NOT DESIGNED to be flushed.

Attends®
.....PREMIER.....

800.4.Attend (800.428.8364) : www.attends.com

©2019 Attends Healthcare Products. All Rights Reserved / 46699 / V1 / 2.2019

Domtar
personal care