

# BD medication preparation and administration with BD SafetyGlide™ Needle




Mountain View Medical Supply  
5076 W. 58th Ave.  
Arvada, CO 80002  
303-455-1300 or 800-873-7121  
MVMSinc.com

## Important usage guidelines

### Preparation: BD™ Blunt Fill and BD™ Blunt Filter Needle



- 1 Attachment**
- Attach the BD™ Blunt Fill or Filter Needle to any standard luer-lock hypodermic syringe. Twist until firmly seated. Pull the red needle shield straight off.

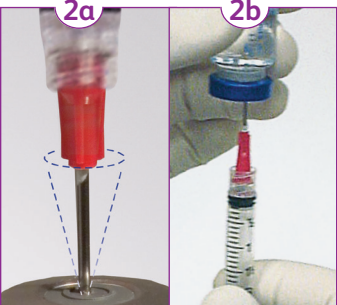

- 2 Aspiration**
- 2a.** Puncture stopper at 90° with the BD™ Blunt Fill Needle to minimize risk of catching internal wall of vial stopper. Note: For glass ampoules use the BD™ Blunt Filter Needle.
- 2b.** Draw up medication in accordance with your institution's protocol.




- 3 Recapping**
- Passively recap BD™ Blunt Fill or Blunt Filter Needle.



- 4 Disposal**
- Detach needle from syringe and immediately discard into an approved sharps collector after single use.

### Administration: BD SafetyGlide™ Needle

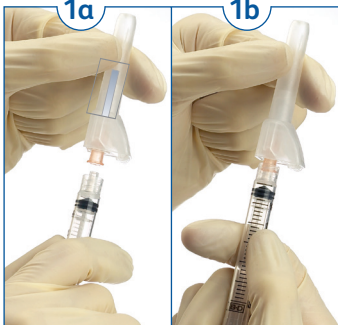

- 1 Attachment**
- 1a.** Attach the BD SafetyGlide™ Needle to any standard luer lock or luer slip hypodermic syringe. Twist until firmly seated to connect to a luer lock syringe.
- 1b.** Pull shield straight off needle to avoid damaging needle point.



- 2 Injection**
- Administer medication in accordance with your institution's protocol. For user convenience, the needle's "bevel up" position is oriented to the safety cover.



- 3 Activation**
- Activate safety mechanism immediately after removal from patient by pushing the lever arm completely forward until needle tip is completely covered.
- Visually confirm that the lever arm has fully advanced and the needle tip is covered. Use one-handed technique and activate away from self and others. After activation or if unable to activate, discard immediately in an approved sharps collector after single use.

#### Remember

- The BD™ Blunt Fill and Blunt Filter Needles are not to be used for skin injections.
- Follow your institution's protocol for aseptic technique.
- Follow your institution's policy for safe disposal of all medical waste.
- The BD™ Blunt Filter Needle is to be used with glass ampoules only.
- The BD™ Blunt Filter Needle is a one-way filter and should be removed once fluid is filtered into the syringe.
- To connect the BD™ Blunt Fill or Blunt Filter Needle to a luer-slip syringe, push firmly.

#### Remember

- To connect the BD SafetyGlide™ Needle to a luer slip syringe, push firmly.
- Follow your institution's protocol for aseptic technique and skin preparation.
- Follow your institution's policy for safe disposal of all medical waste.

#### BD™ Blunt Fill Needle



#### BD™ Blunt Filter Needle



Note hub color difference between blunt fill vs blunt filter needle

bd.com

BD, the BD Logo and SafetyGlide are trademarks of Becton, Dickinson and Company or its affiliates. All other trademarks are the property of their respective owners. © 2019 BD. All rights reserved. BD-1163 (10/19)

