


Hydrofera Blue READY and Hydrofera Blue READY-Transfer Dressings

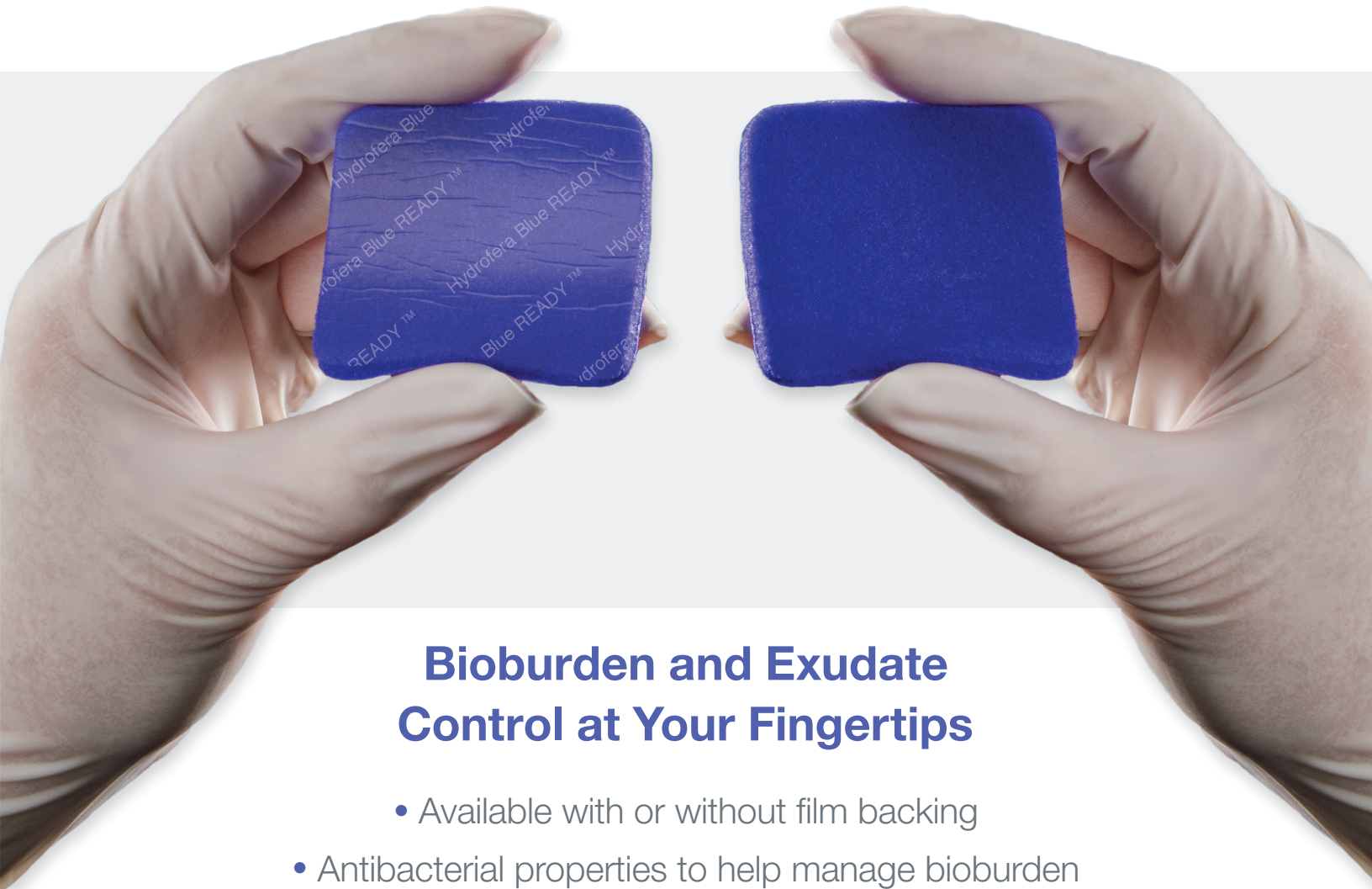


Bioburden and Exudate Control at Your Fingertips

- Available with or without film backing
- Antibacterial properties to help manage bioburden
 - No hydration required prior to application
 - Up to 7-day wear time

Hydrofera[®]

When the Management of Bioburden Is a Priority

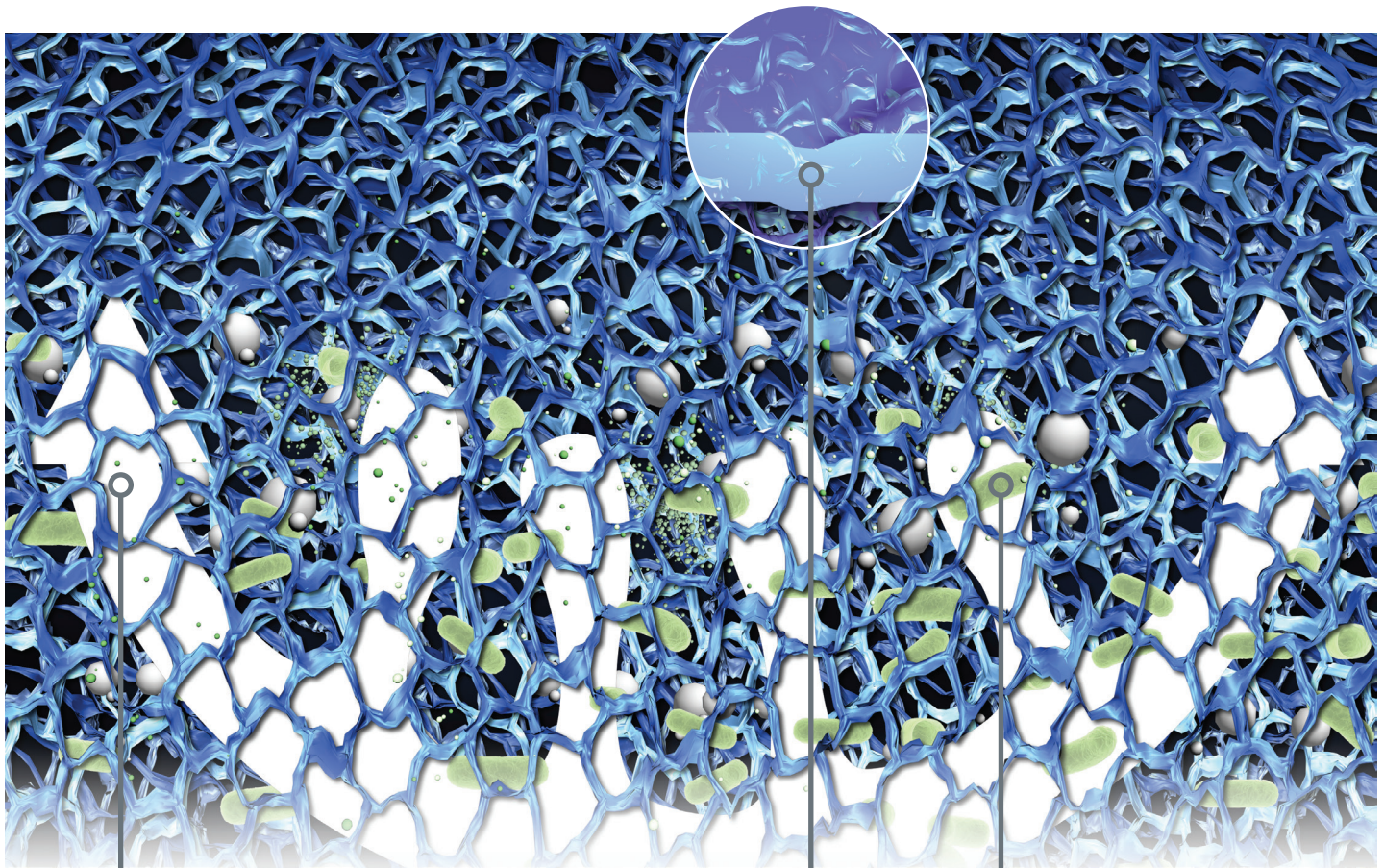

Broad-spectrum antibacterial protection¹

- Provided by two organic pigments, methylene blue and gentian violet
- Noncytotoxic²

Absorption

- Absorbs and retains bacteria-laden exudate

Available with or without waterproof film backing

Hydrofera Blue READY Dressings Provide Broad-Spectrum Antibacterial Protection¹

Bacteria

- *Methicillin-resistant Staphylococcus aureus (MRSA)*
- *Staphylococcus aureus*
- *Staphylococcus epidermidis (coagulase negative)*
- *Pseudomonas aeruginosa*
- *Pseudomonas fluorescens*
- *Escherichia coli*
- *Enterococcus faecalis*
- *Vancomycin-resistant Enterococcus (VRE)*
- *Streptococcus pneumoniae*
- *Streptococcus pyogenes*
- *Klebsiella pneumoniae*
- *Proteus mirabilis*
- *Proteus vulgaris*
- *Enterobacter aerogenes*
- *Enterobacter cloacae*
- *Bacillus subtilis*
- *Yersinia enterocolitica*
- *Serratia marcescens*

Yeast

- *Candida albicans*
- *Candida krusei*
- *Candida glabrata*

Hydrofera Blue READY



- Waterproof film backing
- No secondary dressing required
- Exudate is absorbed and retained
- Up to 7-day wear time

Hydrofera Blue READY-Transfer




- No film backing increases exudate management options
- Exudate is transferred into secondary absorbent dressing
- Suitable for use on partial and full thickness wounds
- Up to 7-day wear time

Performance Data

Absorption Comparison

Hydrofera Blue READY foam dressing has absorption comparable to other antimicrobial foam dressings, based on in vitro testing³

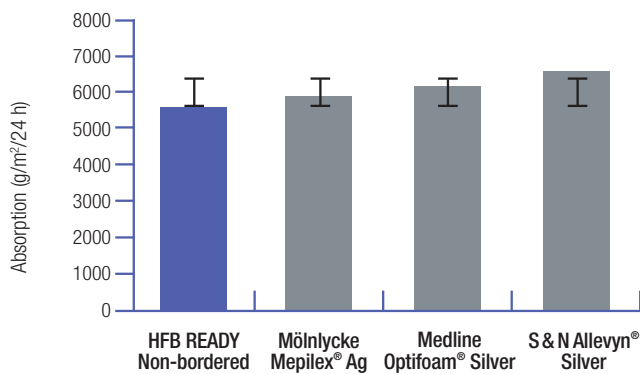


(Error bars represent one standard deviation)

Performance Data

In Vitro Fluid Transfer Study

Hydrofera Blue READY-Transfer has a high rate of exudate transfer to a secondary dressing⁴

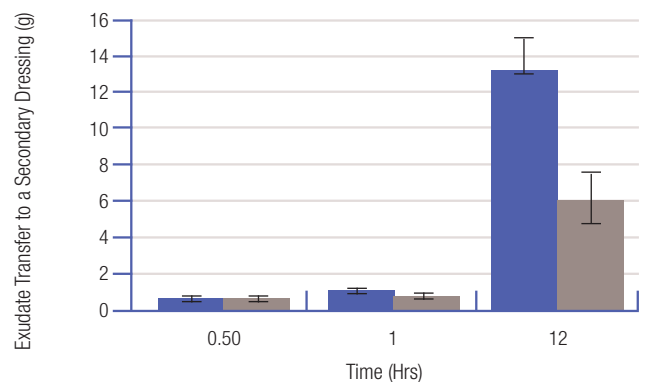

(Error bars represent one standard deviation)

Hydrofera Blue READY-Transfer antibacterial dressings



No hydration required


Stock No	Product Size	Quantity/Box	HCPCS*
Standard Dressings			
HBRT2525	2.5" x 2.5" (6.4 cm x 6.4 cm)	10	A6209
HBRT4050	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HBRT8080	8" x 8" (20.3 cm x 20.3 cm)	10	A6211



No hydration required

Hydrofera Blue READY antibacterial dressings

Stock No	Product Size	Quantity/Box	HCPCS*
Standard Dressings			
HBRS2520	2.5" x 2.5" (6.4 cm x 6.4 cm)	10	A6209
HBRS4520	4" x 5" (10.2 cm x 12.7 cm)	10	A6210
HBRS8820	8" x 8" (20.3 cm x 20.3 cm)	10	A6211

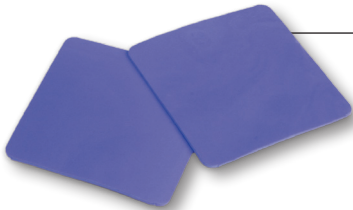

Hydrate before use

Hydrofera Blue CLASSIC antibacterial dressings

Stock No	Product Size	Quantity/Box	HCPCS*
Standard Dressings			
HB2214	2" x 2" (5 cm x 5 cm)	10	A6209
HB4414	4" x 4" (10 cm x 10 cm)	10	A6209
HB6614	6" x 6" (15 cm x 15 cm)	10	A6210
HBT0906	9-mm Tunneling Dressing (1.2 g)	10 pair	A6215
Heavy Drainage Dressings			
HBHD4450	4" x 4" x .5" thick (10 cm x 10 cm x 1.2 cm)	5	A6209
HBHD6675	6" x 6" x .75" thick (15 cm x 15 cm x 1.9 cm)	5	A6210
Dressings With Moisture-Retentive Film			
HBRF4414	4" x 4" (10 cm x 10 cm)	10	A6209
HBRF2258	2.25" x 8" (5.7 cm x 20 cm)	10	A6210
HBRF4475	4" x 4.75" (10 cm x 12 cm) Island Dressing; <i>pad size 2" x 2.75" (5 cm x 6.5 cm)</i>	10	A6212
HBRF2650	2.5" (6 cm) Diameter Ostomy Dressing	10	A6209

Caution: Federal (USA) law restricts this device to sale by or on the order of a physician (or properly licensed practitioner). Prior to the use of this device be sure to read the complete information on how to use this device, including Warnings, Cautions and Instructions for Use and all other package inserts supplied with the product.

For product questions, sampling needs, or detailed clinical questions concerning our products in the US call **860-337-7730**.

Hydrofera®

www.hydrofera.com

Hydrofera Blue Dressings
 Manufactured By:
 Hydrofera, LLC
 340 Progress Drive
 Manchester, CT 06042 USA

1. Instructions for Use
 2. Data on file 2013 – 130107-1CYT
 3. Data on file, EN 13726-1; 2002.
 4. Data on file 2017 - TR-00148
- *HCPCS are for reference only and subject to change.

Hydrofera Blue and Hydrofera Blue READY are trademarks of Hydrofera, LLC.
 All other trademarks are the property of their respective owners.
 Made and printed in the USA.
 ©2018 Hydrofera, LLC.
 002-0518