

# HOW TO ASSEMBLE & INSTALL YOUR



## HOME BED RAIL WITH LEGS

Item # 6094

Capacity not to exceed 300 lb




**NO tools required**  
**Quick installation**

**Easy to follow**  
**step-by-step**  
**instructions**

Mountain View  
Medical Supply  
5076 W. 58th Ave.  
Arvada, CO 80002  
303-455-1300  
800-873-7121  
MVMSinc.com

### WHAT'S IN THE BOX

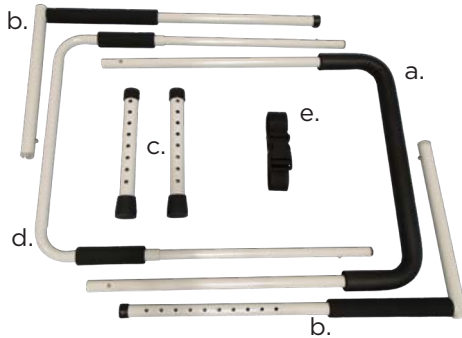

- a. (1) Handle bar
- b. (2) L-shaped bars
- c. (2) Adjustable legs
- d. (1) End bar
- e. (1) Securing strap



### ASSEMBLY INSTRUCTIONS

#### 6094 - Home Bed Rail with Legs

- 1 Attach the two (2) L-shaped bars to the end bar (**Figure 1**), ensuring holes match outward push button direction. Ensure each push button is fully engaged after hearing it “click” into place.

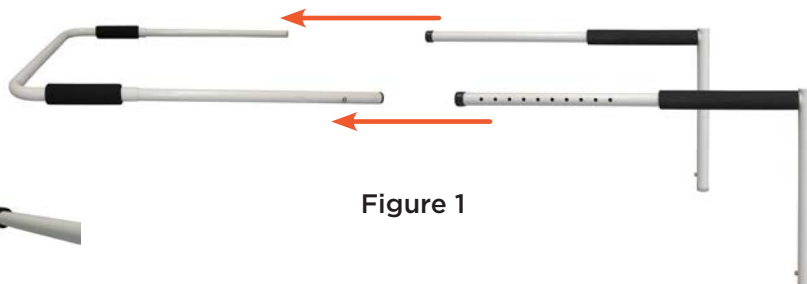


Figure 1



Figure 2

- 2 Attach two (2) adjustable legs to the assembled frame (**Figure 2**). Ensure push buttons are fully engaged.

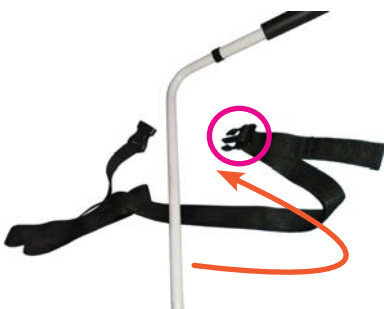


Figure 4

- 3 Attach handle bar to the L-shaped bars (**Figure 3**). Ensure push buttons are fully engaged.

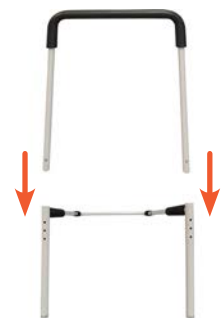


Figure 3

- 4 Thread securing strap (male end first) through the bottom of the bed rail and buckle together (**Figure 4**).

# INSTALLATION INSTRUCTIONS

## 6094 - Home Bed Rail with Legs

- 1** Depress the push buttons on the outside of the end bar and adjust the length of the rail to half the width of the bed (**Figure 1**). Extend securing strap across the bed, and position the end bar of the bed rail at the desired position (**Figure 2**).



Figure 1



Figure 2



Figure 3

- 2** Insert the rail between the mattress and boxspring or platform (**Figure 3**).

- 3** Push the rail until it sits flush against the bed and the remaining length of securing strap is accessible (**Figure 4**).



Figure 4



Figure 5

- 4** Lift the corner of the mattress and feed the securing strap around the corner (**Figure 5**) and place the strap back on bed.

- 5** Go to the other side of the bed and grasp the securing strap (**Figure 6**). Lift the corner of the mattress and feed the strap around (**Figure 7**).



Figure 6



Figure 7

- 6** Unbuckle the securing strap, and wrap the female end around the bed frame (**Figure 8**). Close the buckle, and tighten strap by pulling on the excess until strap is secure (**Figure 9**). This may need to be done multiple times. Tuck away the excess under the mattress.



Figure 8



Figure 9

- 7** Depress the push button on each leg to adjust the height (**Figure 10**). NOTE: Both legs MUST be level and touch the floor. Ensure push buttons are fully engaged before using.




Figure 10



Figure 11

- 8** Depress the push buttons on the handle bar, under the foam padding, to adjust the handle height (**Figure 11**). NOTE: Both sides must be level. Ensure push buttons are fully engaged before using.

- 9** Your Home Bed Rail with Legs is now ready to use!

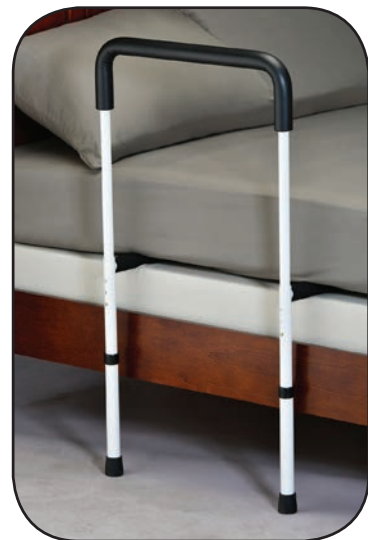

This product is NOT designed or intended to keep a user in bed. If a user runs the risk of unexpectedly rolling out of the bed, a hospital style bed frame is recommended. This product is only designed to allow a user to get in and out of bed with stability and safety.

## **WARNING**

Read all instructions carefully before use. Failure to do so may result in serious injury. **NOVA Medical Products shall have no liability for any damage or injury resulting from improper assembly, installation, and/or use of this product.**

- **ALWAYS** ensure push buttons are fully engaged and protruding from each corresponding hole before using.
- **ALWAYS** ensure handle bar is level and secure before using.
- **ALWAYS** check adjustable legs are fully engaged and stable before using (**model 6094**)
- Assistance and supervision is required for users who are too weak or uncoordinated. **ONLY** use as a stability aid.
- **DO NOT** exceed the weight capacity of **300 lb.**

## **CARE AND MAINTENANCE**

Use a non-abrasive, mild cleaning agent and soft cloth to wipe down surface of the Home Bed Rail. Dry completely before using. **DO NOT** use harsh solvents including lacquer thinner, turpentine, and acetone.

## **WARRANTY INFORMATION**

NOVA stands firm on our commitment to providing the highest quality products, as well as exceptional service. This product was built to precise standards and thoroughly inspected prior to shipment. This warranty represents our confidence in the materials and workmanship of our product.

NOVA extends this warranty only to the original purchaser of this product. The warranty does not extend to any subsequent purchaser or owner. The warranty is void upon any sale or transfer of ownership or use by other person. NOVA warrants its products are free from defects in material and workmanship under **NORMAL** use for the period commencing upon the date of purchase and continuing for the following specified period of time after that date:

Metal Frame	<b>Limited Lifetime Warranty</b>
Non-Metal/Metal parts	<b>One Year Limited Warranty</b> (rubber tips, plastic or metal hardware)

This warranty does not cover problems due to user negligence, misuse of the product, failure to adhere to product instructions, or normal wear and tear. Items that have been subject to negligent abuse, modified without prior written authorization are not covered within this warranty.

For warranty service, please contact the dealer from whom you purchased your NOVA product. In the event that you do not receive satisfactory service, please call us directly at the number on the bottom front of this sheet. Be prepared to indicate, the nature of the defect, the product serial number, and the name and location of where you purchased the product. You will also be required to provide a receipt of your purchase. Do not return any products to our office without prior consent.

Repair or replacement is the only remedy under this limited warranty. This warranty does not include any labor or shipping charges incurred in replacement part installation or repair of product.

**THIS WARRANTY IS EXCLUSIVE AND IN LIEU OF ALL OTHER EXPRESS WARRANTIES, IMPLIED WARRANTIES, IF ANY, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. NOVA SHALL NOT BE LIABLE FOR ANY SEQUENTIAL OR INCIDENTAL DAMAGES WHATSOEVER.**

### **IMPORTANT WARNING**

**FOLLOW ALL DIRECTIONS CAREFULLY BEFORE USE. FAILURE TO DO SO MAY RESULT IN SERIOUS INJURY.**

**CONTACT US!**  
We're here to help


 [info@novajoy.com](mailto:info@novajoy.com)


 1 800 557 6682


[novajoy.com](http://novajoy.com)


*Connect with us!*

[novajoy.com](http://novajoy.com)

 [novamedicalprod](https://www.facebook.com/novamedicalprod)

 [novamedicalprod](https://twitter.com/novamedicalprod)

 [novawalkerstalk](https://twitter.com/novawalkerstalk)

 [NOVA Medical Products](https://www.youtube.com/channel/UC...)

CARE • QUALITY • INNOVATION



THE **NOVA** PROMISE