

# HOW TO INSTALL YOUR

# nova

## Wall Grab Bar

Models: 8012CH-R, 8016CH-R, 8018CH-R, 8024CH-R, 8032CH-R, 8012PC-R, 8016PC-R, 8018PC-R, 8024PC-R, 8032PC-R

Capacity not to exceed 250 lb

Easy to follow  
step by step  
instructions




### TOOLS NEEDED



### WHAT'S IN THE BOX

- a. (1) Grab Bar (chrome or ivory)      c. (6) Plastic Anchors (for tile only)  
b. (6) Mounting Screws                      d. (2) Rubber Seals

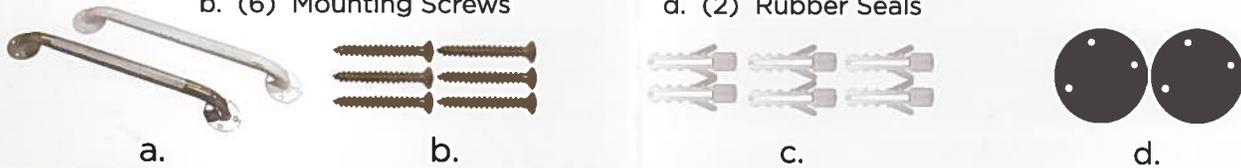

**1** Locate stud behind existing shower wall to determine where the grab bar can be safely installed. Clean portion of wall to be used for mounting

**⚠ WARNING: BOTH GRAB BAR FLANGES MUST BE MOUNTED DIRECTLY INTO WALL STUDS OR USE PLASTIC ANCHORS WHEN NO STUDS ARE AVAILABLE.**

**2** Position mounting flanges and use a pencil to mark where holes need to be drilled. Use a 1/4" drill bit to drill pilot holes through shower wall into wood or metal stud.


**NOTE: If wall is tiled, drill through tile ONLY with a 5/16" drill bit, and then switch to 1/4" drill bit to complete the hole.**

**3** Place rubber seals and mounting flanges against the wall, and align with holes (**Figure 1**). Insert provided mounting screws and screw until tightened.

**NOTE: If wall is tiled, insert plastic anchors into holes before mounting.**



Mountain View Medical Supply  
5076 W. 58th Ave.  
Arvada, CO 80002  
303-455-1300 or 800-873-7121  
MVMSinc.com

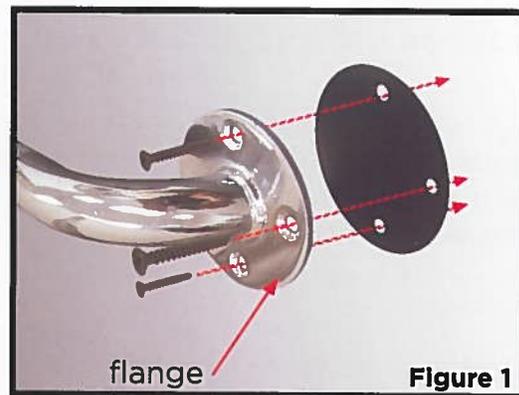

flange

Figure 1

### ⚠ WARNING

Read all instructions carefully and consult with a qualified professional before installing to avoid possible injury. **NOVA Medical Products shall have no liability for any damage to tile or fiberglass walls or injury resulting from improper installations or misuse of this product.**

### HOW TO USE

Grasp grab bar when entering and exiting bathtub or shower. Use to stabilize when needed.

### CARE AND MAINTENANCE

Check regularly to make sure the Wall Grab Bar is securely mounted to the wall. Use a mild detergent and warm water to clean.