# Aneroid Sphygmomanometers

Instructions  
& Warranty

Mountain View  
Medical Supply  
5076 W. 58th Ave.  
Arvada, CO 80002  
303-455-1300 or  
800-873-7121  
MVMSinc.com

**PRESTIGE**  
MEDICAL®

# Thank you for your purchase

Thank you for choosing a Prestige Medical® aneroid sphygmomanometer.

Prestige Medical® is very proud of its fine reputation for supplying precision medical products while delivering the highest level of service to the health care professional.

Your Prestige Medical® aneroid sphygmomanometer has been subjected to the strictest of standards in manufacturing and quality assurance testing. With proper care and maintenance your instrument will provide you with years of dependable service.

***Please review this manual to become familiar with the recommended practices for using and caring for your aneroid sphygmomanometer.***


*Please see page 13 for Warranty details*

# Gauge Calibration

The indicator needle should always rest within the rectangular calibration area. If this is not the case, your gauge needs to be recalibrated to within 3mmHg when compared to a certified reference standard.

***If recalibration is required, please refer to your warranty and return instructions on page 13.***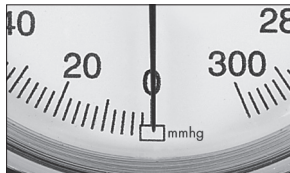


***Note: The gauge is still fully calibrated even if the needle is on the line of the calibration recatgle.***

# Aneroid Sphygmomanometer Anatomy

# Manometer Gauge Anatomy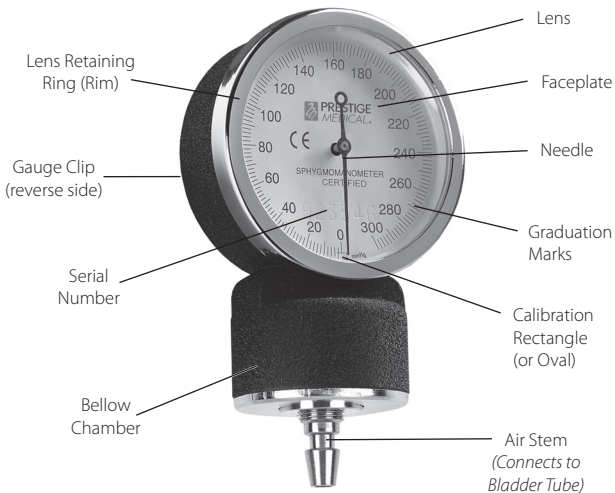


# Cuff Positioning

The cuff should be placed over the bare upper arm with the bladder side over the brachial artery.

Position the bottom edge of the cuff at approximately one inch above the antecubital fold.

Wrap the fabric portion of the cuff around the arm snugly and smoothly, engaging the adhesive system (hook-and-loop). All Prestige Medical® cuffs have index and range markings to ensure an accurate fit.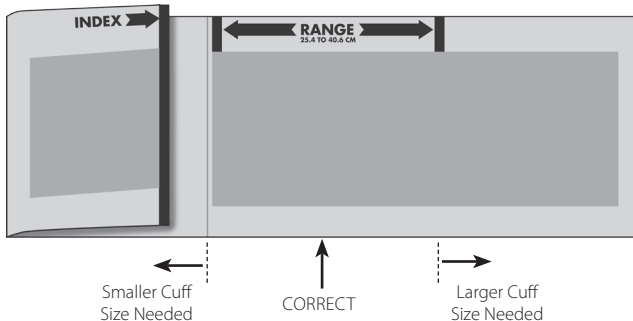

## Position of the Patient

The patient should sit or lie comfortably so that the arm is firmly supported at heart level. The observer should be sure to have a view of the manometer in a direct line to avoid parallax error (i.e. a mis-reading due to skewed visual perception).

## Stethoscope Position


Place the stethoscope chest piece in the antecubital region just below the bottom edge of the cuff using the bell side if it is a combination stethoscope. Do not place the chest piece all the way under the cuff as this will prevent an accurate measurement.

## Cuff Inflation

- Turn the thumbscrew clockwise to close the air release valve.
- Palpate the radial artery while quickly inflating the cuff.
- Inflate cuff to 20 to 30mmHg above the point at which the radial pulse disappears.

***Never inflate the cuff past 300mmHg. This will damage the gauge and void the warranty.***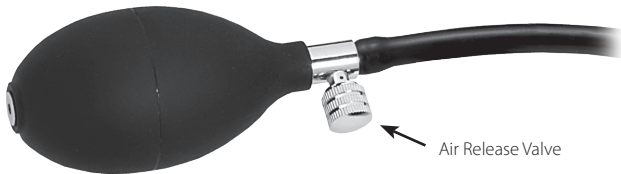

## Cuff Deflation

Open the air release valve (turn counterclockwise), allowing the cuff to deflate gradually at the rate of 2-3mmHg/second.

# Measurement

Record the onset of Korotkoff sounds (systolic pressure) and their disappearance (diastolic pressure).

***For greatest accuracy this instrument should be used in a temperature range between 50°F (10°C) and 104°F (40°C). Use in temperatures outside of that range may result in an inaccurate reading.***

***Do not allow inflated cuff to remain on the patient for longer than 10 minutes. This may cause patient distress and injury related to restricted blood circulation.***

# Cleaning and Maintenance

## **Cuff and Nylon Case:**

Wipe with a damp sponge or cloth. If necessary, wash in cool water with mild soap. Remove bladder prior to washing. Rinse & air dry. Do not iron.

## **Bladder and Bulb:**

Wipe with a damp sponge or cloth. To protect from cracking or discoloring, wipe with a soft cloth moistened with ethanol.

## **Valve and Gauge:**

Clean with a soft dry cloth. NEVER disassemble the gauge!

***When connecting (or disconnecting) the tube to the gauge, simply push (or pull) the tube over the air stem. Do not twist the tube. Twisting the tube may damage the rubber and break the seal.***

*Please see page 13 for Warranty details*

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# Warranty Information

## **Lifetime Warranty on the Gauge:**

If your manometer gauge is out of adjustment during normal use (the needle is outside the calibration rectangle or oval), Prestige Medical® will replace the gauge for the lifetime of the unit. *Excludes some models (see below for details).*

## **5 Year Warranty on the Unit:**

Your aneroid sphygmomanometer is warranted against defects in manufacturing during normal use by the original purchaser for 5 years. *Excludes some models (see below for details).*

---

### **Models: 79, S79**

Includes a 10 year warranty on the gauge and a 5 year warranty on the unit.

### **Models: 70, A1-103, A1-104, A1-105, A1-110**

Includes a one year warranty on both the gauge and the unit.

All returns must first be authorized by emailing customer support at **cs@prestigemedical.com**. In the email, please indicate the problem and the model of your unit. If you would like us to call you, please write your phone number and a good time to call.

# Contact Information

## USA Service Center

Prestige Medical  
Attn: Parts Warranty Program  
8600 Wilbur Ave.  
Northridge, California 91324

## Europe Service Center

NCD Medical  
Attn: Parts Warranty Program  
Dermotstown, Naul  
Co. Dublin, Ireland

## Contact Us :

Email: [cs@prestigemedical.com](mailto:cs@prestigemedical.com)

Web: [www.prestigemedical.com](http://www.prestigemedical.com)

## Follow Us :

/prestigemedical

/prestigemed

/prestigemedical

**PRESTIGE**  
MEDICAL.

**PRESTIGE**  
*MEDICAL*®

PLEASE  
RECYCLE

Aneroids Manual  
03/17