


## When Managing Bioburden is a Priority

- A unique antimicrobial wound dressing that can be used under Ostomy barriers
- Manages bioburden, which helps the body's own immune system restore bacterial balance
- Broad spectrum activity against a wide variety of bacteria and yeast commonly found in wounds
- No hydration required

### Site preparation:

- Prior to the initial preparation of the Hydrofera Blue Ostomy dressing, the wound should be debrided of any necrotic tissue.
- The wound should be cleansed with normal saline or an appropriate cleansing solution.

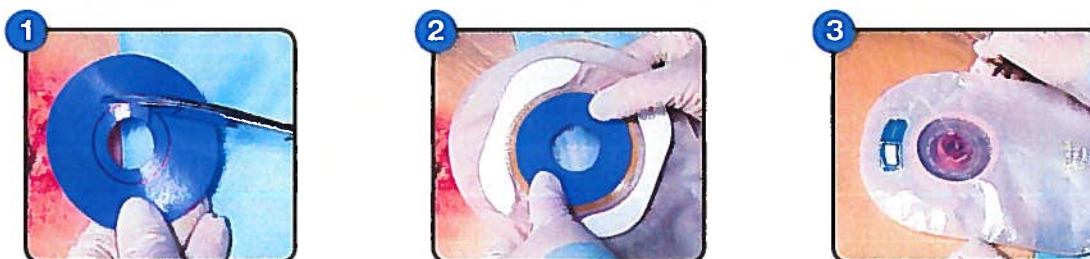

### Application Instructions:

- 1 Trim the inner diameter of the Hydrofera Blue Ostomy dressing to fit around the stoma. If needed, trim the outer edge of the dressing as well, so there is at least 1/2" of the skin barrier extending beyond the outer edge of the dressing.
- 2 Apply the film side of the dressing to the Ostomy skin barrier.
- 3 Apply the skin barrier to the skin

A new Hydrofera Blue Ostomy dressing should be applied when the skin barrier is changed. Hydrofera Blue dressings should be changed every 1-3 days depending on the level of exudate and the condition of the wound.

### For extra security holding the skin barrier in place:

- Consider using an Ostomy belt or skin barrier with tape border
- Consider using a larger skin barrier to ensure coverage over the entire wound area, and so there is adequate skin barrier around the outer circumference of the Hydrofera Blue Ostomy dressing.

## Hydrofera Blue Ostomy Dressing

Stock No	Product Size	Quantity/Box
HBRF2650	2.5" (6 cm) Diameter Ostomy Dressing	10

### Hydrofera Blue Dressings also available:

Stock No	Product Size	Quantity/Box
----------	--------------	--------------

#### Standard Dressings

HB2214	2" x 2" (5 cm x 5 cm)	10
HB4414	4" x 4" (10 cm x 10 cm)	10
HB6614	6" x 6" (15 cm x 15 cm)	10
HBT0906	9 mm Tunneling Dressing (1.2 gm)	10 pairs

#### Heavy Draining Dressings

HBHD4450	4" x 4" x 0.5" thick (10 cm x 10 cm x 1.2 cm)	5
HBHD6675	6" x 6" x 0.75" thick (15 cm x 15 cm x 1.9 cm)	5

#### Dressings with Moisture-Retentive Film (Available in USA only)

HBRF4414	4" x 4" (10 cm x 10 cm)	10
HBRF2258	2.25" x 8" (5.7 cm x 20 cm)	10
HBRF4475	4" x 4.75" (10 cm x 12 cm) Island Dressing pad size 2" x 2.75" (5 cm x 6,5 cm)	10

For product questions, sampling needs, or clinical questions concerning our products, call

**1-860-337-7730**

Hydrofera Dressings manufactured by: Hydrofera, LLC, 340 Progress Drive Manchester, CT 06042, USA

[www.hydrofera.com](http://www.hydrofera.com)